




# **Troubleshooting your BAXJECT® II Needle-less Transfer device**

If water provided did not successfully transfer from the vial, do the following:

1. Place the WATER vial on the bottom.

2. Identify the WHITE and CLEAR portions of the BAXJECT II device.

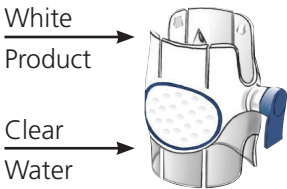

3. Determine which side of the device is connected to the water vial:

If the WATER vial is (correctly) connected to the CLEAR side




Go to  
**SCENARIO A**

If the WATER vial is (incorrectly) connected to the WHITE side



Go to  
**SCENARIO B**

If you are having trouble withdrawing some or all of the mixed solution into the syringe

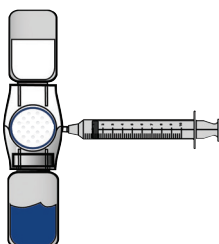

Go to  
**SCENARIO C**



**NOTE:** If these 3 troubleshooting steps fail to address the problems you are having, please contact your doctor or local hemophilia treatment center.

**BAXJECT II PRODUCT INFORMATION**

Rx Only. Always follow the specific instructions given by your healthcare provider regarding use of this device. The BAXJECT II device is intended for use with a single vial of product and is for single use only. If using more than one vial of product, the mixed contents of more than one vial may be drawn into the same syringe. However, a separate BAXJECT II device is needed to mix each additional vial of product.

# Scenario A


The WATER vial is CORRECTLY connected to the CLEAR side of the BAXJECT II device.

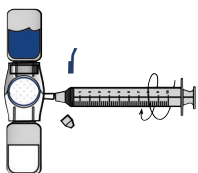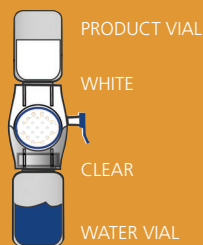

Turn the device (with the vials attached) over so that the WATER vial is on top.


Remove the caps from the device and the syringe.

Connect the syringe to the device.

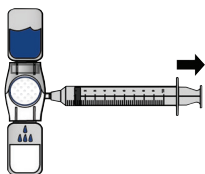

Pull the plunger back SLOWLY to withdraw any air from the vials.


This will force water into the PRODUCT VIAL.

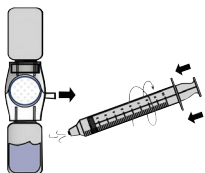


Disconnect the syringe from the device.

Press the plunger to expel any air from the syringe.

Reconnect the syringe to the device.

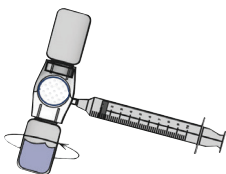

Swirl gently until the product is completely dissolved. **Do not shake.**

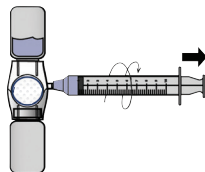



Turn the device (with the vials attached) over so that the mixed solution is on top.

SLOWLY withdraw the prescribed amount of mixed solution into the syringe.


Disconnect the syringe.

# Scenario B

The WATER vial is INCORRECTLY connected to the WHITE side of the BAXJECT II device.

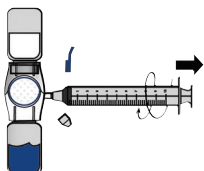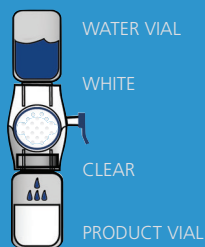


First, make sure the WATER VIAL is on the bottom.  
Remove the caps from the device and the syringe.  
Connect the syringe to the device.  
Pull the plunger back SLOWLY to withdraw any air from the vials.

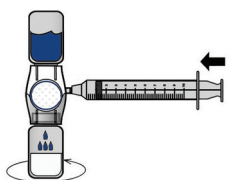

Turn the device (with the vials attached) over so the WATER vial is on top.

SLOWLY press the plunger to inject all the air back into the vials. This will force water into the bottom vial.


Swirl gently until the powder is completely dissolved. **Do not shake.**

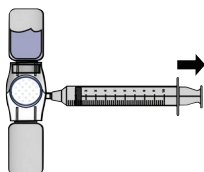

Turn the device (with the vials attached) over so that the mixed solution is on top.

SLOWLY withdraw all the air into the syringe by pulling the plunger back.


This draws the mixed solution into the bottom vial.

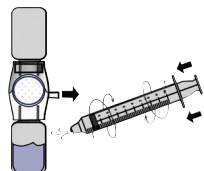

Disconnect the syringe.

Press the plunger to expel all the air from the syringe.

Reconnect the syringe to the device.

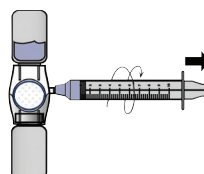


Turn the device (with the vials attached) over so that the vial containing the mixed solution is on top.

SLOWLY pull back the plunger to withdraw the prescribed amount of mixed solution into the syringe.


Disconnect the syringe.

# Scenario C


There is trouble withdrawing some or all of the mixed solution into the syringe.

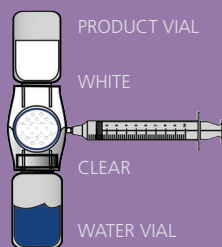


First, make sure that the PRODUCT vial is on top.

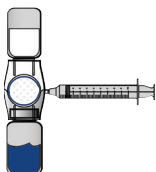


If you have withdrawn some mixed solution into the syringe, but cannot withdraw the full volume, inject all of the mixed solution in the syringe back into the device. Turn the device (with the vials attached) over so that the WATER vial is on top.

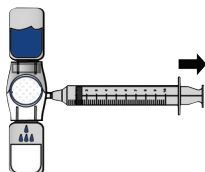


If you cannot withdraw any mixed solution into the syringe, turn the device (with the vials attached) over so that the WATER vial is on top.

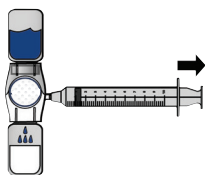


Pull the plunger back SLOWLY to withdraw any air from the vials.  
This will force water into the PRODUCT vial.

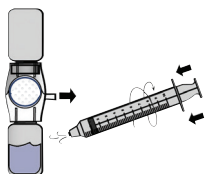

Disconnect the syringe from the device.  
Press the plunger to expel any air from the syringe.  
Reconnect the syringe to the device.

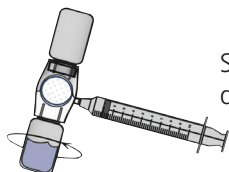

Swirl gently until the product is completely dissolved. **Do not shake.**

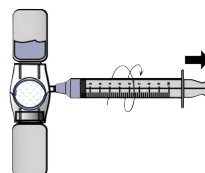

Turn the device (with the vials attached) over so that the mixed solution is on top.  
SLOWLY withdraw the prescribed amount of mixed solution into the syringe.  
Disconnect the syringe.

Copyright ©2019 Takeda Pharmaceutical Company Limited. Lexington, MA 02421. 1-800-828-2088. All rights reserved.

TAKEDA and the TAKEDA logo are trademarks or registered trademarks of Takeda Pharmaceutical Company Limited.

BAXJECT is a trademark or registered trademark of Baxalta Incorporated, a Takeda company.

S50665 10/19

