




Troubleshooting  
your BAXJECT® II  
Needle-less  
Transfer device

# If water did not successfully transfer, do the following:

1. Place the WATER vial on the bottom

2. Identify the WHITE and CLEAR portions of the BAXJECT II device

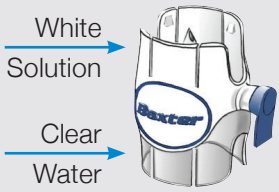

3. Determine which side of the device is connected to the water vial:

If WATER vial is (correctly) connected to CLEAR side




Go to **SCENARIO A**

If WATER vial is (incorrectly) connected to WHITE side



Go to **SCENARIO B**

If you are having trouble withdrawing some or all of the mixed solution into the syringe

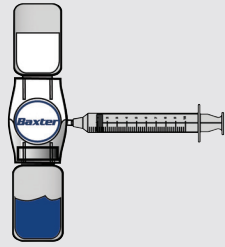

Go to **SCENARIO C**

**NOTE:** If these 3 troubleshooting steps fail to address the problems you are having, please contact your doctor or local Hemophilia Treatment Center.

## BAXJECT II PRODUCT INFORMATION

Rx Only. For safe and proper use of this device, please refer to the instructions for use. The BAXJECT II device is intended for use with a single vial of product and is for single use only. Therefore, reconstitution and withdrawing a second vial into the syringe requires a second BAXJECT II device.

# Scenario A


The WATER vial is CORRECTLY connected to the CLEAR side of the BAXJECT II device

SOLUTION VIAL

WHITE

CLEAR


WATER VIAL

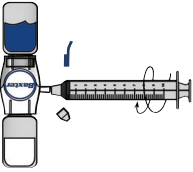

Turn the device (with vials attached) over so that the WATER vial is on top.


Remove the caps from the device and the syringe.

Connect the syringe to the device.

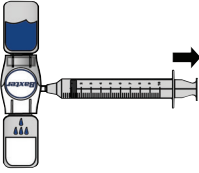

Pull the plunger back SLOWLY to withdraw any air from the vials.


This will force water into the PRODUCT vial.

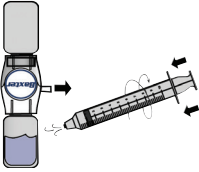


Disconnect the syringe from the device.

Press the plunger to expel any air from the syringe.

Reconnect the syringe to the device.

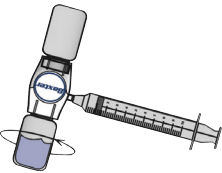

Swirl gently until the product is completely dissolved.

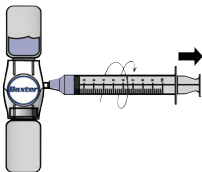



Turn the device (with vials attached) over so that the PRODUCT vial is on top.

SLOWLY withdraw the prescribed amount of mixed product into the syringe.

Disconnect the syringe.

# Scenario B

The WATER vial is INCORRECTLY connected to the WHITE side of the BAXJECT II device

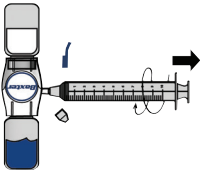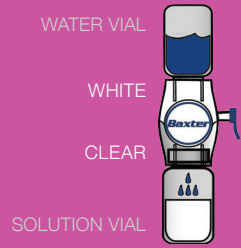


Keep the WATER vial on the bottom.

Remove the caps from the device and the syringe.

Connect the syringe to the device.


Pull the plunger back SLOWLY to withdraw any air from the vials.

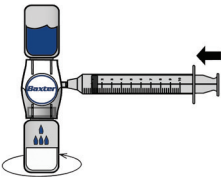

Turn device (with vials attached) over so the WATER vial is on top.

SLOWLY press plunger to inject all air back into the vials. This will force water into the bottom vial.


Swirl gently until powder is completely dissolved.

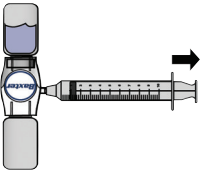

Turn the device (with vials attached) over so that the PRODUCT vial is on top.

SLOWLY withdraw all air into the syringe by pulling the plunger back.


This draws mixed solution into the bottom vial.

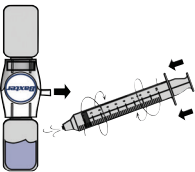

Disconnect the syringe.

Press the plunger to expel all the air from the syringe.

Reconnect the syringe to the device.

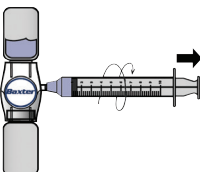


Turn the device (with vials attached) over so that the vial containing mixed solution is on top.

SLOWLY pull back the plunger to withdraw the prescribed amount of mixed solution into the syringe.


Disconnect the syringe.

# Scenario C

There is trouble withdrawing some or all of the mixed product into the syringe into the syringe

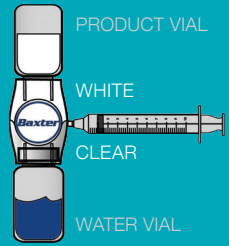


First, make sure that the PRODUCT vial is on top.

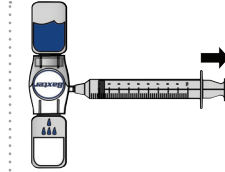


**If you have withdrawn some solution into the syringe, but cannot withdraw the full volume,** inject all of the mixed solution in the syringe back into the device. Turn device (with vials attached) over so that the WATER vial is on top.

**If you cannot withdraw any solution into the syringe,** turn the device (with vials attached) over so that the WATER vial is on top.

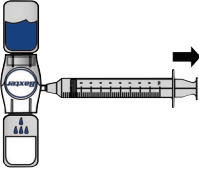

Pull the plunger back **SLOWLY** to withdraw any air from the vials.


This will force water into the PRODUCT vial.

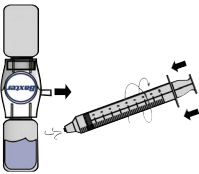


Disconnect the syringe from the device.

Press the plunger to expel any air from the syringe.

Reconnect the syringe to the device.

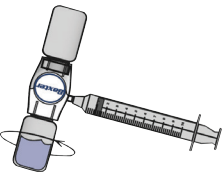

Swirl gently until the product is completely dissolved.

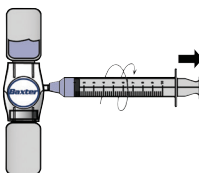

Turn the device (with vials attached) over so that the PRODUCT vial is on top.

**SLOWLY** withdraw the prescribed amount of mixed product into the syringe.

Disconnect the syringe.

©2017 Shire US Inc., Lexington, MA 02421.  
All rights reserved. 1-800-828-2088.

SHIRE and the Shire Logo are registered trademarks of  
Shire Pharmaceutical Holdings Ireland Limited or its affiliates.

BAXJECT is a trademark or registered trademark  
of Baxalta Incorporated, a wholly owned, indirect  
subsidiary of Shire plc.

S26956 1/17

